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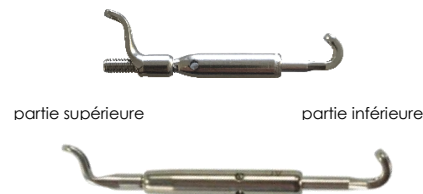
Dispositif médical

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
Date de fabrication

Reference and description of items concerned:

SP2RSC	Adjustable PUL 2 propulsion system - MINI (pair)
SP2C	Adjustable PUL 2 propulsion system - SHORT (pair)
SP2L	PUL 2 adjustable propulsion system - LONG (pair)

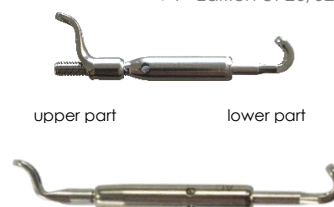Thank you for purchasing our PUL 2 system. **Please read these instructions carefully.** The materials used in the manufacture of the system have a perfect biological tolerance, validated within the framework of regulations relating to medical devices. Sold clean but not sterile, it is recommended that the cleaning rules in force in dental laboratories and surgeries be applied before use. Please observe the storage instructions.

Intended use: intended for the orthodontic treatment of malocclusions in children and adults

Technical information :

Used to make a removable "PUL 2" orthodontic appliance consisting of two resin trays. A pair of systems (identical connecting rods) is fixed to the framework tubes using support rods.

Clinical benefit :

Obtaining Class I

Clinical indication :

- Orthodontic treatment of Class II malocclusions in growing children.
- Snorlight: mandibular advancement orthosis for the treatment of snoring and mild sleep apnoea in adults.

Please note: Use is limited to orthodontic treatment. Use in patients is the responsibility of the practitioner.

Contraindication:

Not suitable for people allergic to Nickel and Cobalt

Laboratory

ASSEMBLY: We recommend that you read the manufacturing protocol from PUL Concept.

The assembly consists of connecting the lower splint to the upper splint (previously manufactured according to the PUL Concept protocol) with the PUL 2 system (ref SP2RSC/SP2C/SP2L) and attachments (ref TBS0.9) on an articulator in the corrected position (according to the practitioner's specifications).

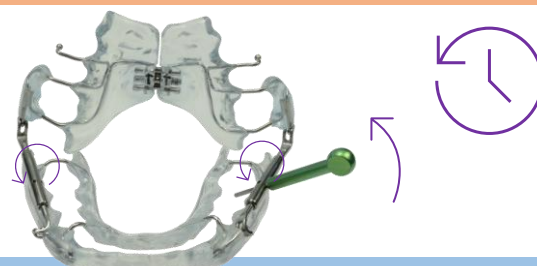
- Place the fastener in the part of the system provided for this purpose and insert the whole assembly into the gutter reinforcement tube, respecting the distance allowing the system to hinge freely (1mm), then fold the fastener over and cut off the excess.
- In this way, attach the upper part of the system to the upper gutter and the lower part of the system to the lower gutter.

Practitioners

ACTIVATION: It is recommended to have attended the clinical training on the PUL concept with Dr CALLABE.

The PUL 2 system is activated by inserting an activation key (ref CABR) into the opening in the middle of the PUL 2 system.

To activate the system, turn by unscrewing counter-clockwise; 3 complete turns = 1mm activation (Activation data sheet available on our website).

Patients

DEVICE PORT :

The device should be worn for 12 to 14 hours a day. The device should not be used for eating, brushing your teeth, playing wind music (flute, trumpet, etc.) or for sport, where there is a risk of breakage or loss.

To activate the retractors, follow the practitioner's instructions

CLEANING :

Before and after each use, clean the appliance under running water with a toothbrush (different from a toothbrush) to prevent tartar build-up.

PRECAUTIONS :

When you remove the appliance, always place it in its box to protect it. Never put it in a handkerchief or paper towel, as it may be thrown in the bin. For canteen meals, remember to remove the device, place it in its box and put it in the school bag so that it doesn't get lost. In the event of a cheek injury, slip a rolled compress between the cheek and the metal system. Do not place the device within the reach of a dog, as it could bite and break it.

Any serious incident involving a medical device from the Pul Concept® range must be reported immediately to your healthcare professional and to PUL Concept on +33 2 54 75 54 80 Email: pulconcept@pulconcept.com; and to the competent healthcare authority in your country.

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