

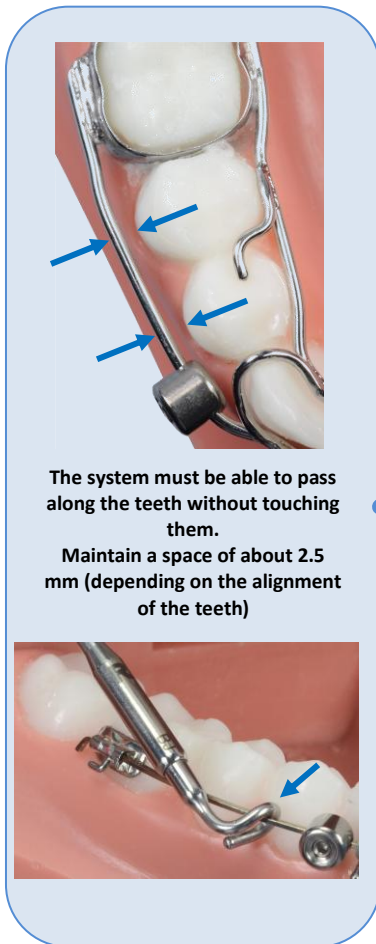



**PUL'Fix with palatal bar
and double lower arch**

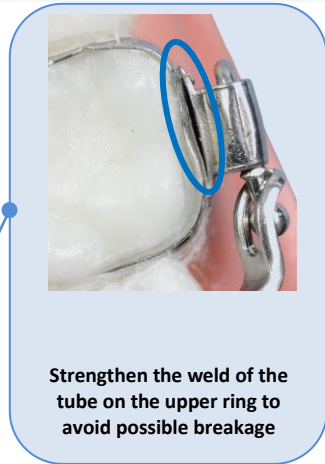


The system must be able to pass along the teeth without touching them.
Maintain a space of about 2.5 mm (depending on the alignment of the teeth)

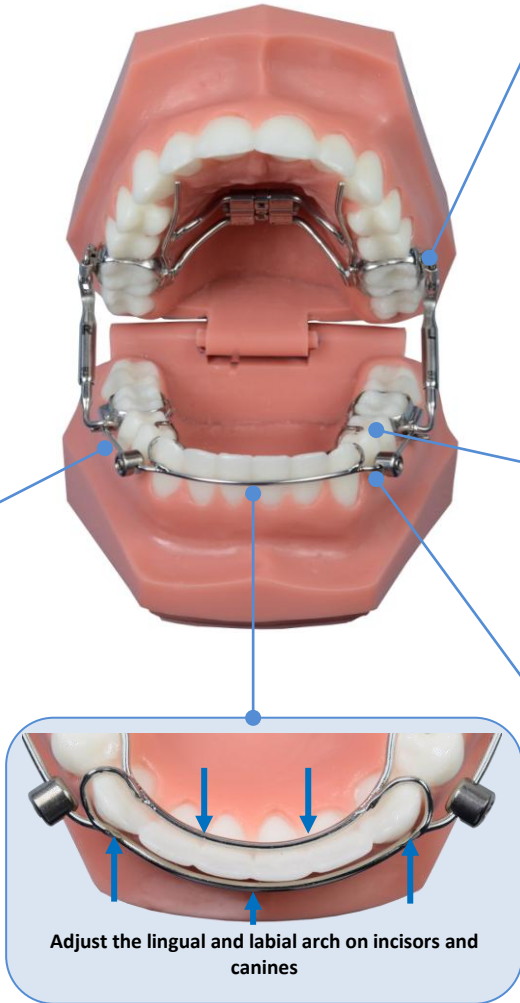


Strengthen the weld of the tube on the upper ring to avoid possible breakage

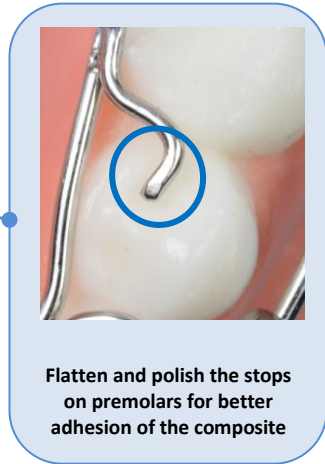

Flatten and polish the stops on premolars for better adhesion of the composite

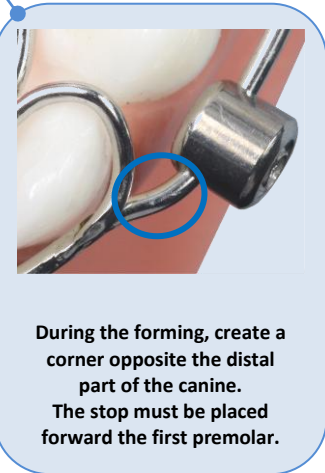

During the forming, create a corner opposite the distal part of the canine.
The stop must be placed forward the first premolar.

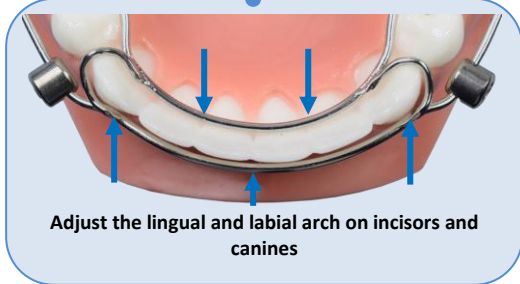

Adjust the lingual and labial arch on incisors and canines

**PUL Concept holds training days on PUL'Fix:
contact@pulconcept.com**

Other videos and PUL'Fix instructions on: www.pulconcept.com & youtube.

MADE IN FRANCE